


# Pro Skin Positioning Sling SPU



Pro-skin  
Gentle  
Positioning




The unique design of the Pro Skin Positioning Sling incorporates DermaTherapy material which decreases the risk of pressure ulcers in clients. The sling has elastic bands on each corner in order to better fit onto the clients' bed. The eight loops each have tree incorporated loops for individual levels of lift height depending on the clients' needs.

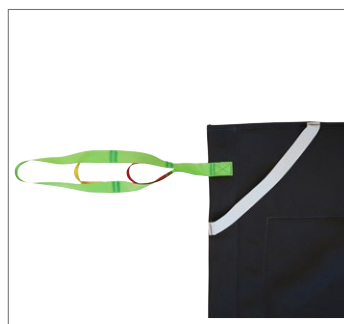

## Human Care positioning slings

Our positioning slings are available in reusable or single patient use models. They are designed to assist a caregiver to safely and to securely perform safe patient handling during intended or sudden positioning/repositioning, turning and lateral transfer with a patient in the supine position.

Used together with a 2-point hanger bar carers are enabled to reposition a client in bed; over to the side or up and down. Used together with a 4-point hanger bar the sling can also assist transfers during situations like 'bed to bed', from 'floor to bed' and from 'bed into a seated position in chair'.

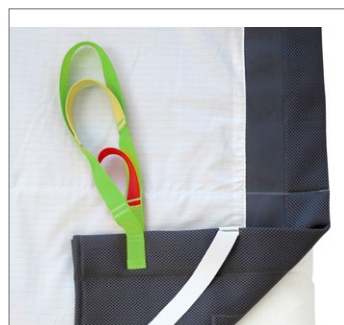

## High quality material

The Soft Air Mesh material allows moisture and heat to transfer efficiently from the patient which allows the sling to be left behind the patient for a period of time after the transfer is complete. The DermaTherapy® material which is incorporated into this positioning sling creates a cleaner, drier and hypoallergenic experience. The DermaTherapy material efficiently decreases the risk for pressure ulcers.

## Easy Care and Easy Fit

Our slings are manufactured to the highest quality standards. The Pro Skin Positioning Sling features eight reinforced loops (6 body/2 head) and high strength nylon webbing around the sling to make them strong enough to enable safe positioning of active clients.



Walking Aids



Healthcare Beds



Lifting Solutions



Hygiene



Convertible chairs

## Accessories



### Extension Loops

Used to extend lift straps in order to alter lifting position or to hold leg parts together. Assorted lengths. Part no.:21180-\*



### Hanger Bar for 4-point Suspension

4-Point - for HeliQ or HeliQ PT  
Article number: 92938



### Hanger Bar for 4-point Suspension

4-Point - Attach to leg hook on Altair/Roomer S  
Article number: 55619

## Lifting Situations



### Use to/from a bed

Suitable for transfers and repositioning to and from a bed. Can be left behind patient for a period of time.



### Use to/from a chair

Suitable for transfers and repositioning to and from a chair. Can be left behind patient for short period of time.



### Single Patient Use Model


Dispose after use.  
Intended for short term use by a single patient.



### Use to/from the floor

Suitable for transfers from the floor.

## Care and Features

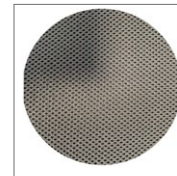

### Soft Air Mesh

Lightweight, cool and breathable material. Suitable for sensitive skin.



## Technical Specifications

Part Number	Weight Capacity	A	B
NSB-451-DT	385 kg / 850 lbs	109 cm / 42,9"	218 cm / 85,8"

Head end

99031, Rev 3, EN, Product sheet, Pro Skin Positioning Sling, Oct 24

[www.humancaregroup.com](http://www.humancaregroup.com)

### HUMAN CARE SWEDEN (HQ)

Årstaängsvägen 21B  
117 43 Stockholm  
Phone:- +46 8 665 35 00  
Fax:- +46 8 665 35 10  
info.se@humancaregroup.com

### HUMAN CARE UNITED STATES

8006 Cameron Road, Suite K  
Austin, TX 78754  
Phone: +512.476.7199  
Fax: +512.476.7190  
info.us@humancaregroup.com

### HUMAN CARE CANADA

10-155 Colonnade Road  
Ottawa, ON K2E 7K1  
Phone: +613.723.6734  
Fax: +613.723.1058  
info.ca@humancaregroup.com

### HUMAN CARE NEDERLAND

Elspeterweg 124  
8076 PA Vierhouten  
Phone: +31 577 412 171  
Fax: +31 577 412 170  
info.nl@humancaregroup.com